



Memoreg® 2 Memosil® 2

The experts for bite registration and special indications.

More satisfaction for you, your dental technician and your patients.

Memoreg 2 and Memosil 2 are our versatile A-silicone-based experts for bite registration and special indications and can be used in prosthetics, dentistry and implantology. They have a high final hardness and can be taken out of the mouth without risk of breakage. The automatic mixing system saves time and reduces risk of failure.

Choose satisfaction and patients' ease and work with either:

- » The pink coloured Memoreg 2 for all standard bite registrations.
- » The transparent Memosil 2 for special indications.

Giving a hand to oral health.



KULZER
MITSUI CHEMICALS GROUP

Memoreg® 2 and Memosil® 2

Your warrantors for exact fitting.

Memoreg® 2

Highly filled, pink-coloured A-silicone for all standard bite registrations which can be applied easily and directly in the mouth. It is best used for standard bite registrations for crown/bridge work, facebow registrations and the fixation of gothic arch registrations.

With Memoreg 2 you and your dental technician benefit from:

- **Optimal thixotropy:**

Its distinct thixotropic properties increase the flow of the material under pressure, and after pressure is released it becomes stable. Furthermore, the material does not drip during the application and can be built up easily.

- **Easy identification of bite-through areas:**

Thanks to the intense colour, you will immediately recognise where the patient has bitten through the material.

- **Comfortable handling:**

The material can easily be removed from the mouth without breaking or tearing. Thanks to its final hardness of 90 Shore A, the dental technician can articulate the models correctly in relation to one another. It is easy to trim and grind with burs, scissors and scalpel. And you are under no time pressure with further processing steps thanks to its dimensional stability.



Source: © Kulzer

Memosil® 2

Transparent A-silicone for special indications. Its transparency allows optical control when positioning aids (e.g. X-ray measuring sphere). Light-curing is possible for filling and fixative materials through placeholders with Memosil 2. It is best used for the occlusal stamp for light-curing posterior composite, for anterior stamp for provisionals, fillings and veneers, as fixation material for X-ray measuring sphere when planning implants or as a transfer matrix for bracket constructions on the plaster model.


With Memosil 2 you and your dental technician benefit from:

- **Optical control:**

Memosil 2's transparency facilitates the positioning of utilities and allows light-curing of composites and fillings through the material.

- **More hygiene:**

Due to its direct and sterile application, processing of Memosil 2 is clean and hygienic.

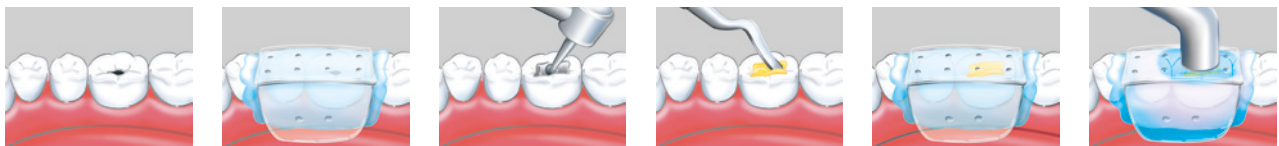

Source: © Kulzer

In addition, both products mean more comfort for your patients. Thanks to their short time in the mouth and the neutral taste and odour, the patients are more at ease. And you save time, as they are directly applied into the patient's mouth.